# CONTROL SAMPLES

ENGLISH / US

for Microdialysis Analysers.

Ref: 8010201

Assayed quality control samples for Microdialysis Analysers.

Also required

Microvials Ref: P000001

Pipette or disposable syringe with a 0.9x38 mm needle or similar, for filling the vials.

Optional

A small centrifuge able to generate a centrifugal force of about 2000 g is optional.

## Intended use

The Control Samples are intended to be used as assayed Quality Control Samples for microdialysis analysers.

These controls should be assayed according to local quality assurance programs, the target concentrations should only be used as a guidance. Use of other control material has not been evaluated.

## Content

5 vials 8010403 Control Sample Low of 5 mL

5 vials 8010205 Control Sample Elevated of 5 mL

For complete information, refer to the "Technical Manual for the 600 Microdialysis Analyser", the ISCUS or ISCUS<sup>®</sup> User Manual

## Procedure

1. Remove and discard the large stopper at the wide end of the microvial.
2. Using a pipette or disposable syringe, fill the vial with 50 – 100 µL of control solution.
3. Remove the air from the narrow end of the vial, preferably using a small centrifuge (30 s at 2000 g).
4. On the 600, click the Analyse Control Sample button  and follow the instructions on the screen. On ISCUS, place the vial in the position labelled CS.
5. To display the results on the 600, click the View Control samples Button . On ISCUS, click the button labelled "Control Samples".
6. Satisfactory level of performance is achieved when the analyte values for the controls are within the "Acceptable Control Range".

## Stability

Unopened vials with control sample are stable until expiry date when stored at +2 to +8 °C. Open vials are stable up to 2 weeks at +2 to +8 °C.

### Symbol declaration

-  For in vitro diagnostic use
-  Last day of use
-  Lot number
-  Storage temperature
-  Consult instructions for use
-  The product meets EU directive for IVD (98/79/EC)

### Caution:

Do not pipette by mouth. Exercise the normal precautions required for handling laboratory reagents.

Opened vials should be handled with care to avoid risk for evaporation and/or contamination.

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