

- For bolt implantation
- “Gold Tip” visible on CT
- For early detection of brain ischemia
- Reinforced shaft

70 Microdialysis Bolt Catheter

This sterile, single use catheter is minimally invasive and designed for implantation in brain tissue through the PTO 2L bolt kit from Raumedic or the IM3 Bolt Kit Integra. The catheter has a 130 mm long shaft and a membrane length of 10 mm. The shaft has a reinforced stainless steel section measuring 16.5 mm in length. This reinforcement is located where the compression screw of the bolt is to be tightened.

The dialysing membrane of the catheter has been especially developed to achieve optimal diffusing characteristics. This allows a high recovery of substances from the extracellular fluid into the catheter.

The “Gold tip” makes the catheter visible on CT


The tip of the catheter contains a gold thread. The “Gold tip” is visible on CT-scanning and makes it possible to locate the exact position of the catheter.

Clinical solution for early detection of ischemia


The 70 Microdialysis Bolt Catheter is connected to a Microdialysis Pump. The outlet tube extends to a holder for a microvial that collects the dialysis sample.

The microvials are usually changed every hour and the samples are transferred to and then analyzed in the ISCUSflex Microdialysis Analyzer. This provides an ideal clinical solution for early detection of ischemia in the brain.

Parts of the 70 Microdialysis Bolt Catheter

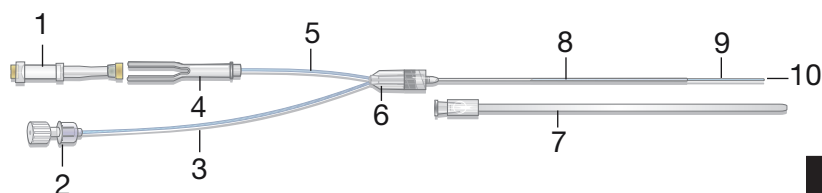

1. Microvial (polystyrene + santoprene)
2. Luer lock connection to syringe (polycarbonate)
3. Inlet tube (polyurethane)
4. Vial holder (polycarbonate)
5. Outlet tube (polyurethane)
6. Liquid cross with luerlock connection (ABS)
7. Protection tube (polyethylene)
8. Shaft (polyurethane) with a 16.5 mm stainless steel re-inforcement
9. Dialysis membrane (polyamide)
10. Gold thread within the catheter tip

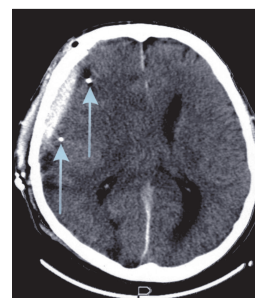

The distal part of the catheter has a gold thread (3 x 0.13 mm) within the catheter tip, which makes the catheter location in the tissue visible on CT

Technical information

	MATERIAL	LENGTH mm	Ø mm
shaft	polyurethane	130	OD 0.9
membrane	polyamide	10	OD 0.6
inlet tube	polyurethane	600	OD 1.0
outlet tube	polyurethane	220	OD 1.0
membrane cut-off 20,000 Daltons			

Ordering information

	Ref. No.
70 Microdialysis Bolt Catheter 4/pkg	P000131

Accessories

	Ref. No.		Ref. No.
Tunneling needle	P000055	106 Microdialysis Pump, 1pc	P000003
Forceps	P000056	107 Microdialysis Pump, 1pc	P000127
Microvials 250/pkg	P000001	106 Pump Syringe 20/pkg	8010191
Microvial Rack 12/pkg	P000028	Battery 6V	8001788
Microvials in rack, Sterile 12x4	P000154	Pump kit for brain tissue	8003791
Perfusion Fluid CNS 10x5mL	P000151		



CE marked according to the Medical Device Directive, 93/42/EEC

STERILE

Sterilized by β -radiation



Storage temperature: 4-25 °C



Single use only



Shelf life: 2 years from the manufacturing date

MDialysis AB Hammarby Fabriksväg 43 SE-120 30 Stockholm, Sweden Tel: +46-8-47010 20 Fax: +46-8-47010 55 www.mdialysis.com

US Office: 73 Princeton Street, North Chelmsford, MA 01863 USA Tel: (866)868-9236 (978) 251-1940 Fax: (978) 251-1960 E-mail: usa@mdialysis.com