Instructions for use: Perfusion Fluid CNS DEXTRAN

GB

INTENDED USE

The fluid is pumped through the 71 High Cut off Brain catheter during microdialysis in brain tissue.

USAGE

1. Open the Perfusion Fluid CNS DEXTRAN vial.
2. Fill the 106 Syringe with maximum 2.5 mL of room-tempered Perfusion Fluid CNS DEXTRAN.
3. Remove any air bubbles in the syringe
4. Connect the inlet tubing of the 71 High Cut off Brain Microdialysis Catheter to the 106 Syringe
5. Place the 106 Syringe in a 106 or a 107 Microdialysis Pump
6. Close the lid to start the pump

For more detailed instruction please read the Instruction for use for the 71 High cut off Brain Microdialysis catheter and 106 or 107 Microdialysis Pump.

CONTRAINDICATIONS & RISKS

- Patients with known hypersensitivity to Dextran.

PRECAUTIONS

- This device is sterile unless the vial/bottle has been opened or damaged
- If there is a suspicion that the fluid has become unsterile prior to usage, it shall not be used

TECHNICAL INFORMATION

Ref. No. 8050151, Perfusion fluid CNS DEXTRAN. Package: Glass vials, 2 x 5 mL

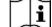
Contents:

NaCl 147 mmol/L, KCl 2.7 mmol/L, CaCl₂ 1.2 mmol/L, MgCl₂ 0.85 mmol/L

Total Chloride contents: 153.8 mmol/L

Dextran MW 500 kDa 3%

SYMBOL EXPLANATION

	Last date of use (YYYY-MM-DD)	CE 0413	Fulfills the Medical Device Directive 93/42/EEC.
	LOT number		Sterilized by use of aseptic processing technique
	Catalogue number		Storage temperature (15-25 °C)
	Single use only		This device is sterile unless the package has been opened or damaged.
	See instructions for use		Manufacturer

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