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## PYRUVATE

REF: P00063

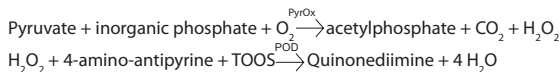
Colorimetric method for the quantitative determination of pyruvate in Microdialysates.

Also required:

Calibrator A. REF: P00057

## Measuring principle

Pyruvate is enzymatically oxidized by pyruvate oxidase (PyrOx). The hydrogen peroxide formed reacts with N-ethyl-N-(2-hydroxy-3-sulfopropyl)-m-toluidine (TOOS) and 4-amino-antipyrine. This reaction is catalyzed by peroxidase (POD) and yields the red-violet colored quinonediimine. The rate of formation is measured photometrically at 530 nm and is proportional to the pyruvate concentration.

Linear range Normal: 10 - 1500 µmol/L  
Low: 10 - 300 µmol/L

## Reagents:

1. Reagent: 5 bottles of lyophilisate
2. Buffer: 5 bottles of 6 mL

Reagent sufficient for 5 x 350 determinations.

Reagents are stable up to expiry date when stored at +2 to +8°C.

## Preparation and stability of solution

1. Unscrew the cap with the membrane from the reagent bottle. Remove and discard the rubber stopper.
2. Transfer the contents of the buffer bottle to the reagent bottle.
3. Fasten the cap with the membrane on the reagent bottle, without Rubber stopper.
4. Dissolve contents completely by gently turning the bottle upside-down at least ten times. Let the reagent stand and equilibrate in room temperature for at least 30 minutes prior to use.

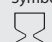
Reconstituted reagent is stable for five days in the instrument.

	Component	Concentration in test solution
Pyruvate reagent	4-amino-antipyrine	0.3 mmol/L
	Tiamine pyrophosphate	0.2 mmol/L
	FAD	10 µmol/L
	Pyruvate Oxidase	> 0.2 kU/L
	Peroxidase	> 0.8 kU/L
	Ascorbate Oxidase	> 10 kU/L
Pyruvate buffer	Citrate buffer, pH 6.1	100 mmol/L
	Potassium dihydrogenphosphate	10 mmol/L
	MgCl <sub>2</sub>	10 mmol/L
	TOOS	1.5 mmol/L
	Sodium azide	0.3 g/L

Sample material Microdialysates	Calibration Use Calibrator A. REF: P00057
------------------------------------	--

## For in vitro use only

## Symbol declaration:

	Last day of use
	Lot number
	Storage temperature
	See instructions for use
	In vitro diagnostic reagent
	The product meets EU directive for IVD (98/79/EC)

## WARNING:

Do not pipette by mouth. Exercise the normal precautions required for handling laboratory reagents.

The buffer contains Sodium Azide. Avoid ingestion or contact with skin or mucous membranes. In case of skin contact, flush affected area with copious amounts of water. In case of contact with eyes or if ingested, seek immediate medical attention.

Sodium Azide may react with lead and copper plumbing, to form potentially explosive azides. When disposing of such reagents, flush with large volumes of water to prevent azide build up. Exposed metal surfaces should be cleaned with 10% sodium hydroxide

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#### Quality Control:

It is recommended that the system be controlled using Control Samples. These controls should be assayed according to local quality assurance schemes. The use of other control materials has not been evaluated. Satisfactory level of performance is achieved when the analyte values for the controls are within the "Acceptable Control Range" published in the Packet Insert with the controls.

Additional information is found in the Technical Manual for the Microdialysis Analyzers.

Manufactured by:  
M Dialysis AB  
Hammarby Fabriksväg 43  
SE-120 30 • Stockholm • Sweden  
Tel: +46-8-470 10 20  
Fax: +46-8-470 10 55  
E-mail: info@mdialysis.com  
www.mdialysis.com

USA office:  
73 Princeton Street  
N.Chelmsford • MA 01863 • USA  
Phone: +1 978 251 1940, +1-866-868-9236  
Fax: +1 978 251-1960  
E-mail: usa@mdialysis.com