
## Instructions for use Splitable Introducer SI-2

### INTENDED USE

The Splitable Introducer SI-2 is intended for introducing Microdialysis Catheters into subcutaneous adipose tissue and resting skeletal muscle, or into hepatic tissue during open surgery.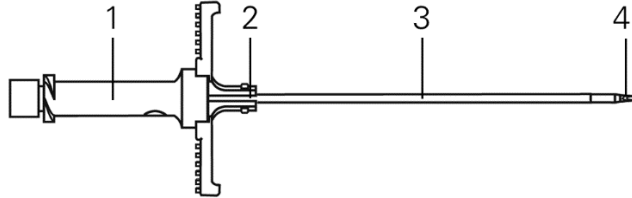

### PRODUCT DESCRIPTION & TECHNICAL INFORMATION


	REF 8010343 Length
1. Handle, cannula (polycarbonate)	—
2. Handle, splitable tube (polytetrafluoroethylene)	—
3. Splitable tube, (polytetrafluoroethylene) OD 1.5	74 mm
4. Cannula (stainless steel) OD 1.0	77 mm

### USAGE

#### Implantation of 61 Microdialysis Catheter in hepatic tissue:

Two persons are required to follow the instructions below

1. Insert the introducer in the middle of the falciform ligament, in direction to segment IV and VIII (2-3 cm above the line between segment V and VIII).
2. Remove the introducer needle carefully while keeping the splitable tube in position. **Warning: do not reinsert introducer needle after with-drawal.** Observe for bleeding. If an extensive bleeding occurs, remove the introducer.
3. Insert the Microdialysis Catheter carefully into the splitable tube. **Do not touch the white membrane of the catheter.**
4. Hold the Microdialysis Catheter in this position with a forceps around the liquid cross. Grasp the handles of the splitable tube (3 hands needed altogether) and pull the handles apart, away from the indwelling catheter, until the tube splits down to its entire length. **After this is accomplished, dispose the plastic strips at once.**
5. Fix the Microdialysis Catheter according to the instructions for the Microdialysis Catheter.

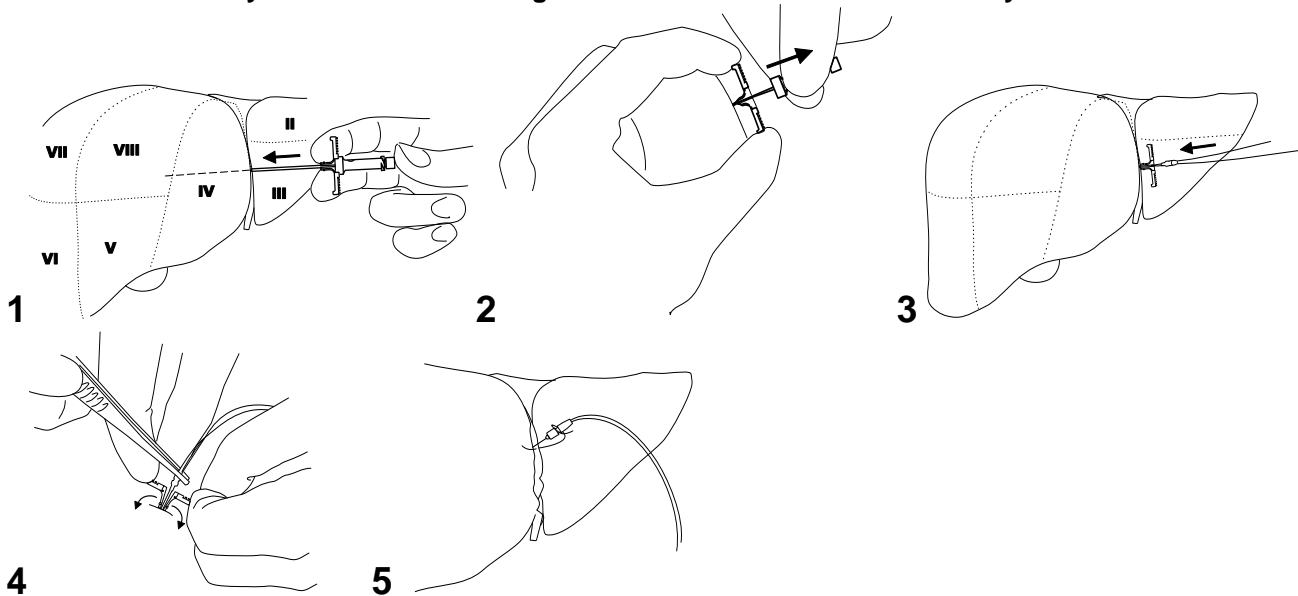For insertion in adipose tissue or resting skeletal muscle tissue, follow the instruction of the 63 Microdialysis catheter.

**NOTE: Used Introducers shall be handled according to the hospital routines for biohazard material.**

**GB**

**SYMBOL EXPLANATION & PRODUCT LIMITATIONS**

	Last month of use (YYYY-MM)
	LOT number
	Catalogue number
	Single use only
	See instructions for use
	Fulfils the Medical Device Directive 93/42/EEC
	Sterilised by β-radiation
	Manufacturer
	Do no use if package is damaged

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