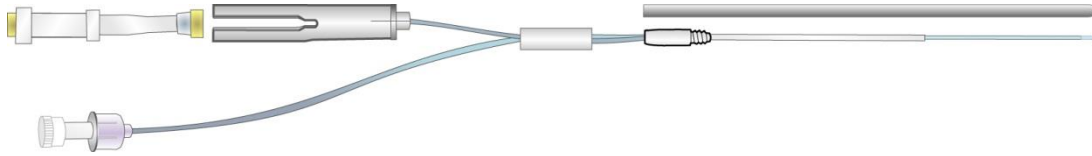
## Instructions for use 63 MICRODIALYSIS CATHETER

### **INTENDED USE**

63 Microdialysis Catheter is intended to perform microdialysis in adipose tissue, resting skeletal muscle or hepatic tissue during open surgery.

### **USAGE**

**NOTE:** Before implantation into adipose tissue the membrane should be pre-wetted. We recommend two ways that this can be done:

- Dip the membrane into sterile perfusion fluid for a few seconds, **or**
- Use the 106 or 107 Microdialysis Pump to flush the catheter with sterile perfusion fluid. Do not remove the protection tubing surrounding the catheter while flushing.

**Use caution so that the catheter does not become unsterile.**

### **Implantation subcutaneously or intramuscular**

The following procedure shall be performed under aseptic conditions by medically trained personnel.

1. Firmly grip the skin with underlying adipose tissue and make a puncture just through the skin using the puncture needle. Break the handles of the Splitable Introducer before insertion to avoid damaging the tissue. Insert the introducer completely to the handle of the splitable tube into the adipose or the resting skeletal muscle tissue.
2. Remove the introducer needle carefully while keeping the splitable tube in position. Do not reinsert the introducer needle after withdrawal. Observe for bleeding. **If an extensive bleeding occurs, remove the introducer.**
3. Unscrew and remove the protective tubing from the catheter, by turning it counterclockwise. **NOTE: Use caution when removing the protective tubing so that you do not damage the membrane at the distal end of the catheter.**
4. Carefully insert the Microdialysis Catheter into the splitable tube. **Do not touch the white membrane of the catheter.**

Two people are required to follow the instructions under number 5 below:

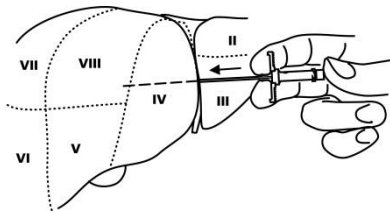
5. Hold the Microdialysis Catheter in this position around the liquid cross. Grasp the tabs of the splitable tube (three hands needed altogether) and pull the tabs apart, away from the indwelling catheter, until the tube splits down to its entire length. After this is accomplished, dispose the plastic strips at once.
6. If needed, carefully anchor the catheter. Please use the fixation devices (12,13) included in the package. Mount the silicon tube (12) on the **shaft**. Place the fixation device (13) over the silicon tube and secure it with tight sutures. Sutures should not be placed directly on the shaft, which may block the flow.
7. If needed the Microdialysis Catheter can also be fixated to the **skin** by sutures around the stopper (6) on the catheter.
8. Cover the insertion site with a protective bandage, e.g. Tegaderm®, OpSite® or similar.
9. **If you use a 106 or 107 Microdialysis Pump:** Connect the inlet tubing to the 106 Syringe, filled with Perfusion Fluid T1, and insert a Microvial into the microvial holder. Place the syringe in the 106 or 107 Microdialysis Pump and close the lid (see more detailed instructions for the pump in the pump manual).
10. **If you use the MD System:** Connect the catheter extension to the 106 Syringe filled with room temperature Perfusion Fluid T1, make sure there are no air bubbles in the syringe. Fill up the extension by hand with Perfusion fluid and connect to the Microdialysis Catheter inlet. Place the Syringe in the MD Cartridge and follow the on-screen instructions (see more detailed instructions, in the MD System manual).


GB

**Implantation in hepatic tissue**

The following procedure should be performed by a surgeon under aseptic conditions. The catheter shall only be implanted and used during open surgery. **NOTE! The catheter shall not be left in the liver after closing of the abdomen as the liquid cross must be placed outside the body.**

1. Break the handles of the Splitable Introducer before insertion to avoid damaging the tissue. Insert the introducer completely to the handle of the splitable tube into the hepatic tissue.
2. Remove the introducer needle carefully while keeping the splitable tube in position. Do not reinsert the introducer needle after withdrawal. Observe for bleeding. **If an extensive bleeding occurs, remove the introducer.**
3. Unscrew and remove the protective tubing from the catheter by turning it counterclockwise. **NOTE: Use caution when removing the protective tubing so that you do not damage the membrane at the distal end of the catheter.**
4. Carefully insert the Microdialysis Catheter into the splitable tube. Do not touch the white membrane of the catheter.

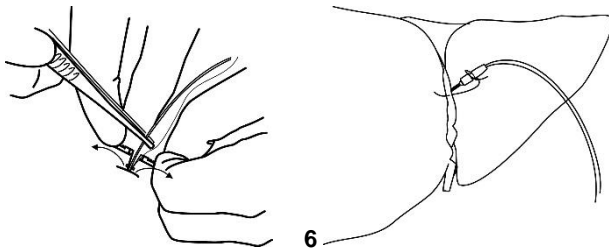- Two people are required to follow the instructions under number 5 below:
5. Hold the Microdialysis Catheter in this position with forceps around the liquid cross. Grasp the tabs of the splitable tube (three hands needed altogether) and pull the tabs apart, away from the indwelling catheter, until the tube splits down to its entire length. After this is accomplished, dispose the plastic strips at once.
6. If needed fix the Microdialysis Catheter to the liver by sutures around the stopper (6) on the catheter.
7. If you use a 106 or 107 Microdialysis Pump: Connect the inlet tubing to the 106 Syringe, filled with Perfusion Fluid T1, and insert a Microvial into the microvial holder. Place the syringe in the 106 or 107 Microdialysis Pump and close the lid (see more detailed instructions for the pump in the pump manual).
8. If you use the MD system: Connect the catheter extension to the 106 Syringe filled with room temperature Perfusion Fluid T1, make sure there are no air bubbles in the syringe. Fill up the extension by hand with Perfusion fluid and connect to the Microdialysis Catheter inlet. Place the Syringe in the MD Cartridge and follow the on-screen instructions (see more detailed instructions, in the MD System manual).

**REMOVAL OF CATHETER**

The catheter is removed by gently pulling it out through the insertion site. **NOTE:** Discontinued/removed catheters shall be handled according to the hospital routines for biohazard material.

**INDICATIONS**

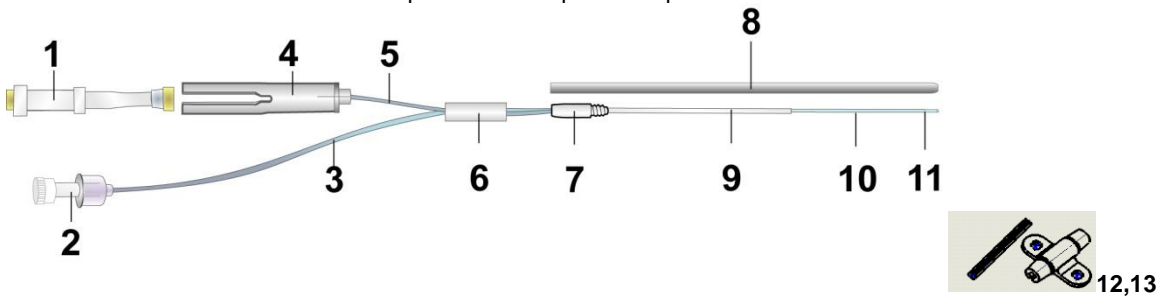
To monitor the patients in a clinical or research setting for the detection of chemical changes, e.g. ischemia, hypoxia or cell damage. Microdialysis shall not be used as the sole basis for diagnosis or therapy.

**CONTRAINDICATIONS & RISKS**

- Do not use the catheter if the mass of the liver tissue is very small.
- Inserting the catheter into the tissue may cause bleeding from damaged vessels.
- The catheter shall not be left in the liver after closing of the abdomen, as the liquid cross must be placed outside the body.
- The 63 Microdialysis Catheter is for single use only. If the device is re-used there is a risk for cross-contamination.

**PRECAUTIONS**

- This device is sterile unless the package has been opened or damaged.
- The 63 Microdialysis Catheter shall only be used together with the accessories described in the table below.
- Be sure to handle the catheter carefully to avoid kinking or other damage, particularly after removal of the protection tube. Avoid contact with the dialysis membrane.
- If any visible damage is observed the catheter shall not be used.
- If there is a suspicion that the catheter has become unsterile prior to insertion the catheter shall not be used.
- The pump syringe connected to the catheter should not manually be flushed since that could damage the dialysis membrane.
- If using with a 106/107 MD Pump: Check that liquid is being pumped through the catheter by inspecting the volume in the microvial **each time** the microvials are changed.
- If using with MD System: Check that there are readings on the screen.
- If using 106/107 MD Pump: If there is no fluid in the collected vial, start a flush on the pump: Open the lid, wait 3 seconds and close it again. Wait for the flush (5 minutes). Check that the tubing's are not kinked and that the microvial holder needle is correctly piercing the microvial membrane. If there is still no fluid in the collected vial, the dialysis membrane might be damaged, and the catheter has to be removed.
- If using with MD System: Check on screen messages.
- Remove the catheter if there is a permanent stop in the liquid flow.

**PRODUCT DESCRIPTION & TECHNICAL INFORMATION**

**63 Microdialysis Catheter**

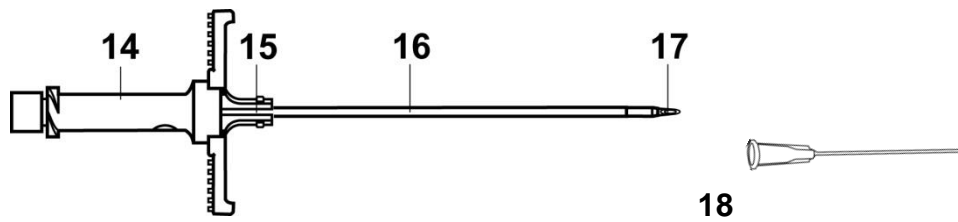
	REF 8010509	REF 8010514
1. Microvial (Polystyrene + Santoprene)	X	X
2. Luer lock connection (Polycarbonate)	X	X
3. Inlet tube (Polyurethane, OD 1 mm)	600 mm	600 mm
4. Vial holder (Polycarbonate)	X	X
5. Outlet tube (Polyurethane, OD 1 mm)	220 mm	220 mm
6. Stopper (Silicone)	X	X
7. Liquid cross (Polysulfone)	X	X
8. Protection tube (Polyethylene)	145 mm	145 mm
9. Shaft (Polyurethane, OD 0.9 mm)	60 mm	40 mm
10. Dialysis membrane (Polyarylethersulphone, PAES, OD 0.6 mm) Cut off: 20 000 Dalton	10 mm	30 mm
11. Gold thread within the catheter membrane tip (OD 0.13 mm, L 3 mm).	X	X
12. Fixation tube (Silicone)	X	X
13. Fixation device (Pebax)	X	X
Dead Volume Time - From membrane to microvial (at 0.3 µl/min)	17 min	17 min

X=Identical parts on both catheters

The distal part of the catheter has a gold thread within the catheter tip, which makes it possible to detect the location of the catheter in the tissue through CT-scanning.

GB

<b>SPLITABLE INTRODUCER SI-2</b>	
14. Handle, Cannula (Polycarbonate)	–
15. Handle, Splitable tube (Polytetrafluorethylene)	–
16. Splitable tube, (Polytetrafluorethylene, OD 1.5 mm)	74 mm
17. Cannula (Stainless steel, OD 1.0 mm)	77 mm
18. Puncture needle (Stainless steel, OD 1.6 mm)	40 mm

**ACCESSORIES**

The 63 Microdialysis Catheter shall only be used with these accessories.

REF	Name
P000034	Perfusion fluid T1 10x5 mL
8010191	106 Syringe 20/pkg
P000154	Microvials in rack, Sterile 12x4
P000003	106 Microdialysis Pump
P000127	107 Microdialysis Pump
8070228	MD System

**CONSUMABLES**

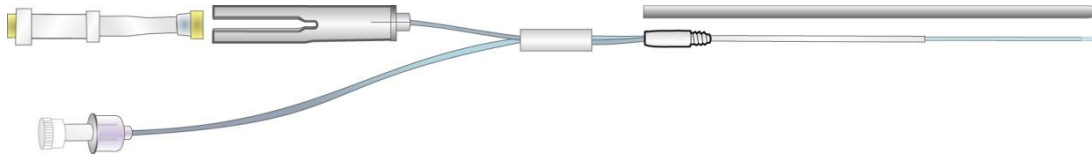
The 63 Microdialysis Catheter should be used with these consumables.

REF	Name
P000001	Microvials 250/pkg

**SYMBOL EXPLANATION & PRODUCT LIMITATIONS**

	Last month of use (YYYY-MM-DD)
	LOT number
	Catalogue number
	Single use only
	See instructions for use
	Fulfills the Medical Device Directive 93/42/EEC.
	Sterilised by β-radiation
	Storage temperature (4-25 °C)
	Manufacturer
	Do not use if package is damaged
Membrane	Membrane material (Polyarylethersulphone, PAES)
Memb. length	Membrane length (mm)
Shaft length	Shaft length (mm)
Outlet	Outlet length (mm)
Inlet	Inlet length (mm)

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