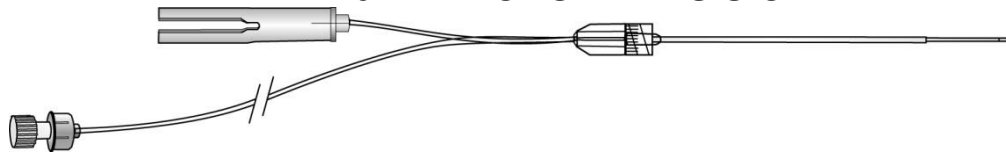
Instructions for use 67 IV MICRODIALYSIS CATHETER

INTENDED USE

67 Intravenous Microdialysis Catheter is a sterile, single use product intended for microdialysis sampling in the peripheral venous system or in Cerebrospinal Fluid (CSF) eg. via an External Ventricular Drain (EVD) for up to three days.

USAGE IN PERIPHERAL VEIN

The following procedure shall be performed by a physician or a nurse under aseptic conditions. The 67 IV Microdialysis Catheter **must not be inserted close to an infusion site as this might influence sampling, it must neither be placed at the bend of the arm as it can be damaged if the patient bends the arm!**

1. Fill the 106 Syringe with a mixture of **Ringer-Acetate and dalteparin sodium (25 IU/mL)**, e.g. Fragmin®, and make sure there are no air bubbles in the syringe. Connect the inlet tubing of the Catheter to the syringe and place it in a 106/107 Pump and close the lid to start the pump.

2. Insert the peripheral venous catheter delivered with the Catheter in a peripheral vein according to normal hospital routines (Fig 1). Observe that the peripheral venous catheter delivered with the Catheter must be used. Flush the PVC with a saline solution.

3. Unscrew the protecting tubing and remove it from the Catheter. Insert the Catheter through the peripheral venous catheter (Fig 2) and fixate it by attaching the Luer-lock of the Catheter to the Luer-lock of the peripheral venous catheter (Fig 3).

WARNING: Be very careful during insertion as the white membrane is fragile. After insertion do not remove the Catheter from the peripheral venous catheter.

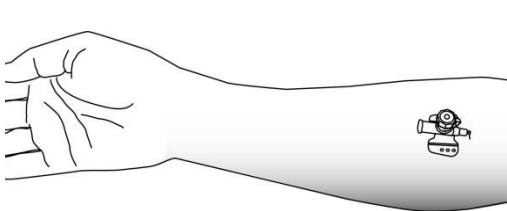
Fig1

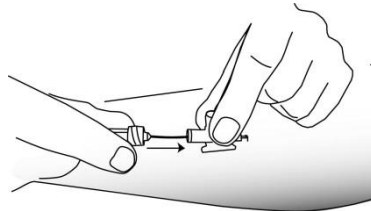
Fig 2

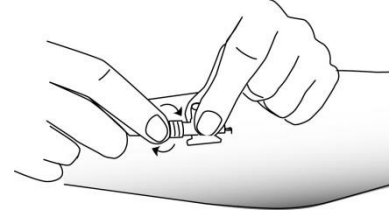
Fig3

4. Attach the pump to the patient by a bandage or similar (Fig 4). Insert a microvial into the microvial holder.

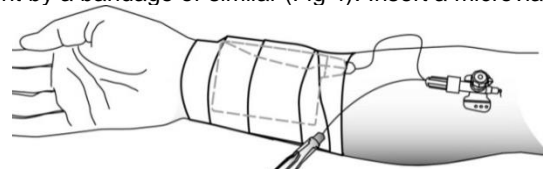
Fig 4

5. Change the microvials regularly. Note that the concentrations of the measured substances are an average of the time measured. If a more transient measurement is required, change the microvial after only a few minutes and analyze it.

USAGE IN CSF DRAIN

The following procedure shall be performed by a physician or a nurse under aseptic conditions.

Fill the 106 Syringe with Perfusion Fluid CNS and make sure there are no air bubbles in the syringe. Connect the inlet tubing of the 67 IV MD Catheter to the syringe and place it in a 106/107 Pump and close the lid to start the pump. Remove the protective tube at the 67 IV MD catheter. Insert and connect the 67 IV Microdialysis Catheter Luer lock to a Stop cock connector; be careful when inserting the Microdialysis catheter so the white dialysis membrane does not get damaged. Connect the Stop cock connector to a CSF Drain out-flow from a CSF drain pump e.g. LiquoGuard®. Change the microvials regularly in the Microvial holder of the Microdialysis catheter. Please note that the peripheral vein catheter (PVC) is not needed when using the 67 IV MD Catheter in CSF Drain.

It is recommended to use the shorter version of the 67 Microdialysis Catheter (Ref 8050090) to ensure that the catheter fits to the Stop cock connector.

Find more information in the Reference Publication: *Microdialysis monitoring of CSF parameters in severe traumatic brain injury patients: a novel approach. Frontiers in Neurology, September 2014.*

REMOVAL OF THE CATHETER

In peripheral vein: The 67 IV Microdialysis Catheter is removed together with the peripheral venous catheter from the vein. **Do not remove the 67 IV Microdialysis Catheter from the peripheral venous catheter when it is in the patient.**

In CSF: The catheter is removed when the drain is removed. **NOTE:** Discontinued/removed catheters shall be handled according to the hospital routines for biohazard material.

INDICATIONS

Monitoring of small substances in peripheral blood or CSF drainage. Microdialysis shall not be used as the sole basis for diagnosis or therapy.

CONTRAINDICATIONS & RISKS

Patients with known hypersensitivity to dalteparin sodium.

The 67 IV Microdialysis Catheter is for single use only. If the device is re-used there is a risk for cross-contamination.

PRECAUTIONS

- Usage in peripheral vein: The 67 IV Microdialysis Catheter shall only be used with the peripheral venous catheter included in the package.
- Usage in peripheral vein: **The 67 IV Microdialysis Catheter must be perfused with a sterile mixture of Ringer's Acetate and dalteparin sodium (25 IU/mL), e.g. Fragmin®, to reduce the risk of blood clotting on the membrane.**
- The peripheral venous catheter is equipped with needle protection in order to reduce the risk of accidental needle stick injuries; however, care must be taken to avoid needle stick injuries to reduce risk of exposure to contaminated blood
- Usage in peripheral vein: Remove the 67 IV Microdialysis and peripheral venous catheter if there are any signs of infection.
- Usage in peripheral vein: Do not place the 67 IV Microdialysis Catheter in the bend of the arm since it can be damaged if the patient bends the arm.
- This device is sterile unless the package has been opened or damaged.
- The 67 IV Microdialysis Catheter shall only be used together with the accessories described in the table below.
- Be sure to handle the 67 IV Microdialysis Catheter carefully to avoid kinking or other damage, particularly after removal of the protection tube. **Avoid contact with the dialysis membrane.**
- If any visible damage is observed the 67 IV Microdialysis Catheter shall not be used.
- The pump syringe connected to the catheter should not manually be flushed since that could damage the dialysis membrane.
- If there is a suspicion that the device has become unsterile prior to insertion it shall not be used.
- Check that liquid is being pumped through the 67 IV Microdialysis Catheter by inspecting the volume in the microvial **each time** the microvials are changed.
- If there is no fluid in the collected vial, start a flush on the pump: Open the lid, wait 3 seconds and close it again. Wait for the flush (5 minutes). Check that the tubing's are not kinked and that the microvial holder needle is correctly piercing the microvial membrane. If there is still no fluid in the collected vial, the dialysis membrane might be damaged and the 67 IV Microdialysis Catheter has to be removed.
- Remove the peripheral venous catheter and the 67 IV Microdialysis Catheter if there is a permanent stop in the liquid flow.
- Do not use the 67 IV Microdialysis Catheter in arterial vessels.
- Usage in CSF: Make sure that the Luer lock connection of the 67 IV Microdialysis Catheter is tightened when connected to the Stop cock
- Usage in CSF: If there is a permanent stop in the flow of the CSF drain, the Microdialysis result might be un-reliable.
- Usage in CSF: Make sure that the Microdialysis catheters white membrane does not pass a tap in the Stop cock with a subsequent risk for damaging the 67 IV Microdialysis Catheter.

ACCESSORIES

The 67 IV Microdialysis Catheter shall only be used with these accessories.

REF	Name
8010191	106 Syringe 20/pkg
P000154	Microvials in rack, Sterile 12x4
P000003	106 Microdialysis Pump
P000127	107 Microdialysis Pump
P000151	Perfusion Fluid CNS (when used in CSF drain)

CONSUMABLES


The 67 IV Microdialysis Catheter should be used with these consumables.

REF	Name
P000001	Microvials 250/pkg

GB

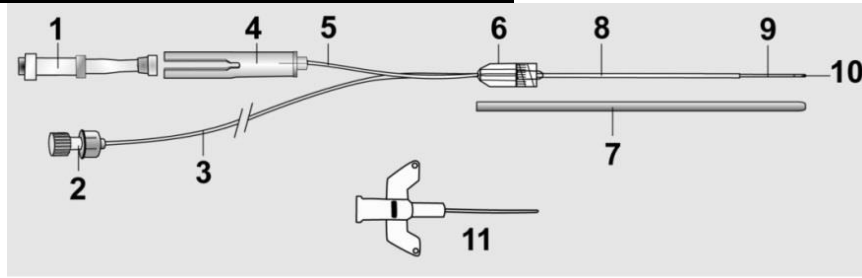PRODUCT DESCRIPTION & TECHNICAL INFORMATION

X=Identical parts on all catheters

	REF 8050090	REF 8050091	REF 8050092	REF 8050093
1. Microvial (Polystyrene + Santoprene)	x	x	x	x
2. Luer lock connection (Polycarbonate)	x	x	x	x
3. Inlet tube (Polyurethane, OD 1 mm)	200 mm	200 mm	200 mm	200 mm
4. Vial holder (Polycarbonate)	X	X	X	
5. Outlet tube (Polyurethane, OD 1 mm)	45 mm	45 mm	45 mm	45 mm
6. Liquid cross (Polysulfone)	x	x	x	x
7. Protection tube (Polyethylene)	145 mm	145 mm	145 mm	145 mm
8. Shaft (Polyurethane, OD 0.9 mm)	46 mm	46 mm	46 mm	130 mm
9. Dialysis membrane (Polyarylethersulphone, PAES, OD 0.6 mm). Membrane cut off: 20 000 Dalton	10 mm	20 mm	30 mm	10 mm
10. Gold thread within the catheter membrane tip (OD 0.13 mm; L=3mm).	x	x	x	x
11. Peripheral Venous Catheter (Polypropylene, ABS, Chrome-nickel Steel, Polyurethane)	18G 1.3 x 32mm	18G 1.3x32mm	18G 1.3x32mm	16G 1.7x50mm
Dead Volume Time - From membrane to microvial (at 0.3 μl/min)	5 min	5 min	5 min	23 min


The distal part of the 67 IV Microdialysis Catheter has a gold thread within tip, which makes it possible to detect the location of the 67 IV Microdialysis Catheter through CT-scanning.

SYMBOL EXPLANATION & PRODUCT LIMITATIONS

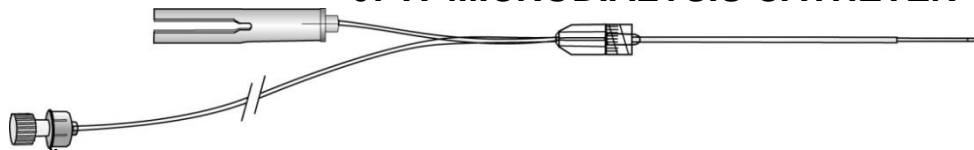	Last date of use (YYYY-MM-DD)
	LOT number
	Catalogue number
	Single use only
	See instructions for use
	Fulfils the Medical Device Directive 93/42/EEC.
	Sterilised by β-radiation
	Storage temperature (4-25 °C)
	This device is sterile unless the package has been opened or damaged.
	Manufacturer
Membrane	Membrane material
Memb. length	Membrane length (mm)
Shaft length	Shaft length (mm)
Outlet	Outlet length (mm)
Inlet	Inlet length (mm)

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