

2026-03-18 Magnus Hedberg

# ISCUS<sup>flex</sup> New Software Version March 2026

- Rev M Version 3.0.11.44



# ISCUS<sup>flex</sup> New Software Version March 2026

- Rev M Version 3.0.11.44



# Overview

- New operating system – Windows 11
- New controlling computer – Surface Pro 11
- New main board
- Software ported from Win CE5
- Built in debug
- Built in ICU Pilot and IscusCollect
- Fast responses in GUI
- AnyDesk for remote support

# Selected defect fixes (TT #340 – TT #393)

- TT #340 New Service Code ICUPILOT to start or show ICUpilot
- TT #344 PowerMonitor. Monitors power
- TT #347 Several service codes added (PACKLOGS)
- TT #348 CPL files may be run (Control Panel files)
- TT #349 Brightness Control. Software control of Screen brightness
- TT #365 Factory reset function requested. Service code FACTORY RESET

# Selected defect fixes (TT #340 – TT #393)

- TT #377 Messages dialog should copy text to Clipboard
- TT #380 Service log entries shall be created when closing the application
- TT #383 Loopback network shall be possible
- TT #384 Using localhost in Settings/Data shall be possible
- TT #386 New service codes DEBUG, DEBUG STOP and DEBUG CLEAN

# Selected defect fixes (TT #340 – TT #393)

- TT #387 New service code DEBUG SCANNER
- TT #388 New service code ISCUSCOLLECT
- TT #389 A simple method to find old data in graphs
- TT #390 Printout for A4/Letter format with CSV option
- TT #391 Printout dialog should have printer icon for printer selection access

# Service codes

- SHOW PACKLOGS - Show packlogs folder in explorer
- PACKLOGS - Packs log files for trouble shooting
- PACKLOGS CLEAR - Clears the packlogs files, only the latest logs are packed
- CLOSE - Closes the running application (ServiceGUI/UserGUI)
- RESTART - Restarts the application

# Service codes

- ICUpilot – To start ICUpilot
- FACTORY RESET – To reset machine parameters
- DEBUG – To start OnlineLogger
- DEBUG STOP – To stop OnlineLogger
- DEBUG CLEAN – To delete old logs
- DEBUG SCANNER – To start measurement curves exploration

- ISCUSCOLLECT – To start IscusCollect for saving data in ICUpilot and relaying data to ER/PDMS

# Service codes

- [D6558-23 P192 ISCUSflex Service codes \(Rev M 3.0.11.44 and later\)](#)